



Freedom Designs, Inc.

Foam-in-Place Seating




Free Form Back Mod-2S

Provides the capability to mold the most difficult client with fixed deformities while providing aggressive positioning with thoracic supports, shoulder retractors, chest vests, and headrests


- Standard includes 1" foam of your choice (excluding pudge).
- Cover is 4" thick in total and has 1/4" Sunmate foam sewn inside the contact surface.
- Stretchable cover fabric is recommended.
- Available in kit (as shown) or in self mixing bags.

1.

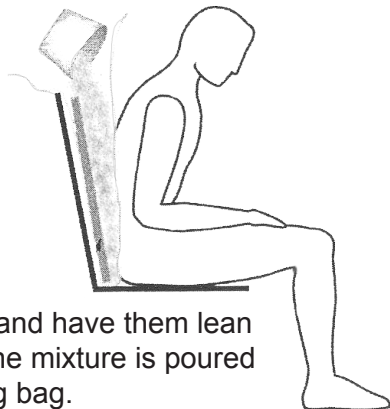

Seat the client and have them lean forward while the mixture is poured into the molding bag.

2.

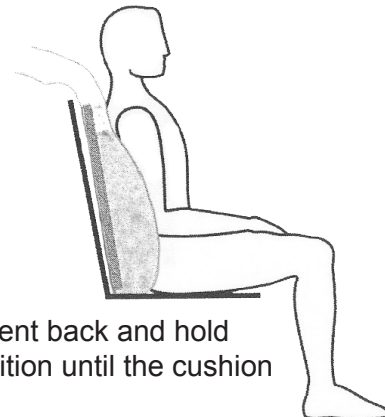

Lean the client back and hold them in position until the cushion cures.



3.



Insert into back for finishing



Kit includes Sunmate chemicals, gloves, safety goggles, bag, and mixing bucket.