

Instructions to Retrofit Pindot ShapeSensor Frame with Narrow Wheelbase Option


The wheelbase of the Pindot ShapeSensor frame can be made narrower by the installation of this retrofit kit. By replacing the main wheels with under-frame casters, the wheelbase will be reduced from 30" wide to 26 1/2" wide. This allows the frame to fit on standard wheelchair ramps.

The retrofit kit consists of the following components:

- Under-frame caster assembly
- 2 Mounting bolts and washers.



Remove the two front wheel mounting bolts and wheels.

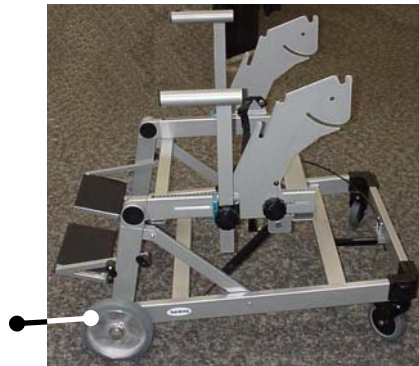

Install the under-frame caster assembly, using the two bolts and washers provided in the kit.



Pindot Customer Service & Technical Support
(800) 451-3553

899 Cleveland St, Elyria, OH 44036

